

INTERNATIONAL SHIPPING BOX

PRODUCT CONTENTS



The following packaging and documentation for international samples only:

- i. One (1) polystyrene box.
- ii. One (1) packaging label (UN3373).
- iii. Eight (8) ice packs that should be placed in the freezer (at -20°C) upon receipt of the box, for a minimum of 24 hours before ready for use.

Please follow the additional packaging and documentation, as well as instructions for collection, transportation and storage of samples for preimplantation genetic testing on the 'Amfira Sample Collection and Transport Kit'.



SUPPORT AND ORDERING INFORMATION

For assistance or to order additional kits, please call +(357) 22 266 888 or email info.genetics@medicover.com. Additional information is available online at www.medicover-genetics.com.




INTERNATIONAL SHIPPING BOX

Please refer to the Amfira Instructions For Use document for more information on sample collection and transport.

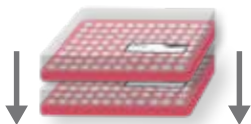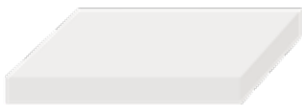

The shipping box can fit 1 or 2 Amfira kits.

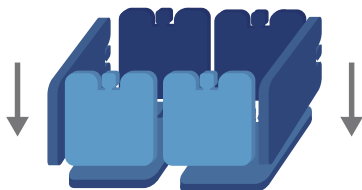


Place 2 icepacks on each long side of the box, 1 on each short side and 2 at the bottom, a total of 8.

The shipping box contains eight (8) ice packs that should be placed flat in the freezer (at -20°C) upon receipt of the box, for a minimum of 24 hours before ready for use.



Shipping Box Front View

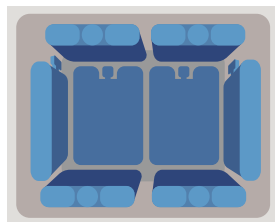

Shipping Box Top View