



# Medicover Genetics

## Sample collection kit

Buccal swab sampling  
for genetic tests



**MEDICOVER**  
GENETICS


Please follow these instructions for **Buccal swab** samples



**1.** Complete the appropriate order form/informed consent form



**2.** Place a barcode on the swab tube and on the order form/informed consent form

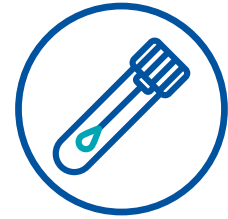

**3.** Unscrew the tube and take out the swab



**4.** Rub the inside of the cheek 5 times, repeat for other cheek



**5.** Place tube in white container for transport



**6.** Place sample in provided padded bag

---

**Send sample and signed forms  
to our laboratory:**

**MVZ Martinsried, Lochhamer Str. 29,  
82152 Planegg, Germany**

*Send sample as soon as possible to begin analysis, however,  
sample is stable at room temperature for a long time*